



Infëkto[®] Cide N Spray

Instant Spray Disinfectant
for Medical Devices

ANTRIX™



Infekto[®] Cide N Spray

Ready-to-use Rapid Spray Disinfectant for Sensitive Medical Devices & Surfaces

ANTRIX[™] products are manufactured as per EN/CE norms in GMP/GLP certified facilities using carefully selected Raw Materials imported from accredited global vendors.

Range of Application:

- For quick disinfection of operation tables, treating areas, dental chairs etc. between two patients.
- For disinfection of high risk areas like Operation Theatres, ICCU, NICU, AKD etc.
- For disinfection of surfaces and objects that cannot be washed with water based disinfectants.
- One product for disinfection of surfaces and sensitive medical equipment, lights, operation tables, cardiac and anaesthetic accessories, airconditioning/airhandling units, computers etc. Also ideal for laboratories & diagnostic centers.
- For effective disinfection in media labs, vaccine & injectable manufacturing units in pharmaceutical industries.

Recommended Guidelines of Use:

- Switch off all electrical / electronic equipment before spraying.
- Spray surface medical devices and lights etc. thoroughly from a distance of approx. 30 cms.
- Spray 20-40 ml. of solutions per sq. meter for instant disinfection, and allow alcohol to dry.

Mode of Application:

- To be sprayed according to label claim or as per recommendations from hospital's infection control committee.
- Spraying should be done with manual sprayer only.
- Spray till surface becomes visibly wet & allow it to dry for 5-10 min for optimum result.
- Do not spray on alcohol resistant surfaces.

Composition:

Active ingredients in 100 gms

- Ethanol 40 gms
- 2-Propanol 10 gms
- Glutaral 50% 0.036 gms
- Benzalkonium Chloride 50% 0.10 gms

ANTRIX[™]

Marketed by:
ANTRIX HYGIENE LIMITED
 803, Panchratna, Opera House, Mumbai 400 004
 Tel : +91 22 4004 9121, Email : info@antrixgroup.com
www.antrixgroup.com

Manufactured under D.L.No. RAJ-2517
 by **ENZI HEALTH CARE**
 C-1/A, Mewar Industrial Area,
 Jhamar Kotra Road, Udaipur - 313 003

Infekto[®] Cide N Spray Offers:

- Ready-to-use formulation with excellent cleaning & disinfection action.
- Broad spectrum of microbicidal, fungicidal & virucidal action.
- Efficacy against MRSA & Mycobacteria (TBC).
- Effectiveness against HBV, HIV, HCV & Rota Virus.
- Disinfection without visible residue.
- Rapid action between 5 seconds and 5 minutes.
- Rapid disinfection for OTs between surgeries.
- Compatible with all types of surfaces - especially for water sensitive surfaces & equipment.
- Simple solution for rapid disinfection of the ICU, NICU, OT, AKD, Dental Unit & Ophthalmologist.

Delivery Units :

20 x 500 ml bottles
 2 x 5 ltrs canister

Your Local ANTRIX[™] Partner :