

# Steam Indicator Tape / Autoclave Tape

## CLASS I Process Indicator Tape for Sterilization

A proven method of positively confirming completed sterilization cycle in both gravity discharge and vacuum autoclaves. Green indicator lines change to black when proper levels of steam pressure and temperature have been achieved. The uniquely printed lines cannot be rubbed off and will not contaminate surgical instruments or dressings.

This easy-to-handle tape adheres on contact and stays in place through live steam pressure. After use, the tape removes quickly and completely, leaving no residue or staining.

This tape is made of a saturated crepe paper printed with green indicator lines that turn to black once autoclave process is completed.

Autoclave tape is the result of exhaustive research and testing to prove its efficiency and unerring accuracy.

This product is sold and used worldwide on a regular basis

### TECHNICAL DATA

Color:	Natural with printed green indicator lines
Backing:	Saturated Crepe Paper
Indicator:	Lead Free Ink
Thickness:	6.0mil (0.15mm)
Adhesion to Steel	30 oz/in (8.3N/25mm)
Tensile strength	20 lb/in (89N/25mm)
Elongation	7%
Standard Sterilization Cycle (with pre-vacuum cycle)	121 °C(250 °F) for 20 minutes 132 to 134 °C (270 to 274°F) for 4 mins

Always consult manufacturer instructions concerning the recommended sterilization cycle.


Meets Canadian Specification.. CAN/CSA-Z11140-1-07 (R2012)

Meets US Specification ANSI/AAMI/ISO 11140-1:2005 (R2010)

Meets European Specification EN ISO 11140-1:2009

### STOCK SIZE

18mm x 55m 48 rolls/case

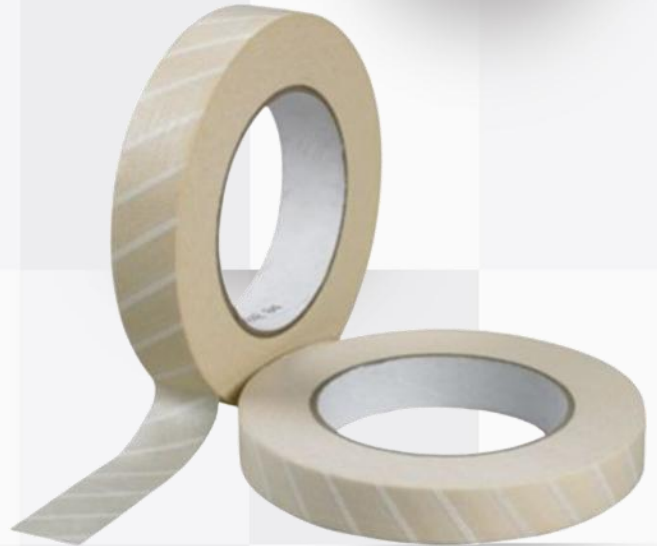

**LATEX  
&  
LEAD FREE**

**BEFORE CYCLE**

**CYCLE IN PROGRESS**

**CYCLE COMPLETED**

**STERIMAXX INC**

*"A reliable partner in Sterilization !!!"*

A/55 Shankesh Bldg, Sanghvi Nagar,  
Mira Bhayander Road, Mira Road (East),  
Thane 401 107 Maharashtra, INDIA.  
Cell: +91 9987 51 4946.

sterimaxx@gmail.com. www.sterimax.com