

Concentrates for surface disinfection

BIGUANID FLÄCHE N

Liquid concentrate for surface disinfection and cleaning

- Strong cleaning power
- Pleasant odour
- Suitable for linoleum and PVC



Product description

BIGUANID FLÄCHE N is an aldehyde-free and phenol-free concentrate for disinfection and cleaning of non-invasive medical devices, medical inventory as well as for all kinds of surfaces.

BIGUANID FLÄCHE N has a broad efficacy and is active against norovirus and has a tuberculocide claim. The user-friendly formulation of BIGUANID FLÄCHE N is based on quaternary ammonium compounds and contains neither aldehydes nor phenols.

The product contains very efficient corrosion inhibitors and has a good material compatibility. It is recommended to be used in areas where next to the high hygienic demand an unpleasant smell must be avoided.

Range of application

Acc. to Biocidal Products Regulation:

Surface disinfection and cleaning of medical inventory and all kinds of surfaces, e.g.

- PVC/linoleum floors as well as acrylic glass
- disinfection and cleaning of bathtubs

Acc. to MDD (Medical Device Directive):

Aldehyde-free disinfection and cleaning of non-invasive medical devices.

Spectrum of efficacy and contact times			5 min	15 min	30 min	60 min	240 min
Bacteria and fungi							
Application recommendation for surface disinfection (4-field test, bactericidal, yeasticidal)*	VAH / EN 16615	with mechanical action, dirty conditions	1 %	0.75 %	0.5 %	0.5 %	0.25 %
Enveloped viruses							
active against enveloped viruses	RKI/DVV	incl. HBV/HIV/HCV		0.5 %			
Non-enveloped viruses							
active against noroviruses (MNV)	EN 14476	clean conditions				1 %	
active against adenoviruses	EN 14476	clean conditions					2 %
active against SV40/papovaviruses	RKI/DVV					1 %	0.5 %
active against rotaviruses	RKI/DVV	clean conditions		0.25 %			
Additional test results							
tuberculocidal (M.terrae)	DGHM	dirty conditions					0.5 %

*including phase 2 step 1 and phase 2 step 2 tests (quantitative suspension tests and practical germ-carrier tests)

Application

Surface disinfection by wiping:

Apply a sufficient quantity of the prepared Biguanid Fläche N working solution to the surfaces to be disinfected. Use mechanical action to completely wet all surfaces (scrub-wipe disinfection). and let dry by air. In the daily routine, the surface can be used directly after the surface is completely dry.

Dry wipes application:

BIGUANID FLÄCHE N can be used with the ONE SYSTEM dry wipes dispenser system for a convenient and user-friendly application with a proven disinfectant efficacy and standing time of up to 42 days and in combination with the DESCO/ECO WIPES dispenser system with a standing time of up to 28 days.



BIGUANID FLÄCHE N

Liquid concentrate for surface disinfection and cleaning

Application notes

Before the first application of **BIGUANID FLÄCHE N** a basic cleaning procedure with an appropriate product such as **HANSA CLEAN SPEZIAL** is recommended.

Dispose of contents/container to approved disposal company or local collection.

For professional use only.

Composition

100 g solution contain:

16 g Alkyl(C12-16)dimethylbenzylammonium chloride.

Contains: Non ionic surfactants 15-30 %, amphoteric surfactants 5-15 %, disinfectants, perfumes

Expert opinions

We have extensive expert reports for all our products. Upon request we are glad to supply you with an expert report portfolio.

Precautionary and hazard statements

Causes severe skin burns and eye damage. Very toxic to aquatic life with long lasting effects. Wear protective gloves/protective clothing/eye protection/face protection. IF SWALLOWED: rinse mouth. Do NOT induce vomiting. IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. IF exposed or concerned: Call a POISON CENTER/doctor.

Product	Single unit	Content	Delivery unit	REF
BIGUANID FLÄCHE N	Bag	40 ml	200	00-201-0004-01
	Dosage bottle	1 L	10	00-201-010
	Bottle	2 L	6	00-201-020
	Container	5 L	3	00-201-050
	Container	10 L	1	00-201-100
	Container	25 L	1	00-201-250
Application aids	Dosage pump	500/1000 ml	1	00-906-005-02
	Dosage pump	2L/5 L/10 L	1	00-906-000-01
	Tap	2L/5 L/10 L	1	00-907-000
Additional supplies	Container opener	-	1	00-909-050

CE 0482



Use disinfectants safely. Always read label and product information before use.

Dr. Schumacher is certified according to DIN EN 13485, DIN EN ISO9001, DIN EN ISO14001, BS OHSAS 18011 and has a validated environment management system according to EMAS and is a member of IHO, VCI, BAH, DGSV and of the DGKH. www.schumacher-online.com

