

STERIKLEEN POWDER

Autoclave & Distiller concentrate powder for cleaning.

Sterikleen is convenient, easy and hassle free way to keep Your equipments free of deposit & scales. Effectively cleans and decalcify the reservoirs, chamber, Heating elements and tubing's in minutes.

For Use In:

- ✓ Table Top Sterilizer.
- ✓ Distill Water Plant.
- ✓ Regular Vertical Autoclave.
- ✓ Heating Coil of Autoclaves.
- ✓ Ultrasonic Cleaners.



Cleaning Procedure:

Table Top Sterilizer:

1. Spread powder (approx 2 tbsp) on the bottom of the normal autoclave chamber.
2. Select regular sterilization cycle with Water & No Vacuum Drying cycle.
3. Remove Solution from reservoir and discard.
4. Repeat steps 2 & 3 without powder.
5. Clean the chamber once it reaches at normal Temp. With damp cloth or sponge.
6. Fill the reservoir with distilled water and sterilizer is now ready to use.

Distill Water Plant:

1. Add 2.5 liters of water & 2 tbsp of powder into chamber. Mix it well.
2. Keep aside upper cover & plug in main power to heat the chamber for approx 15-20 mins.
3. Throw the left over water and wipe the chamber with damp cloth / sponge.
4. Repeat steps 1, 2 & 3 if residue is still visible. However small amount of residue is harmless.
5. Fill fresh water start new cycle.

Regular Vertical Autoclave:

1. It is used for removing lime scale deposit from your autoclave heating element.
2. Mix well 2-3 tbsp of powder into 2 liters of water and pour to inlet valve.
3. Now add water till the heater coil is immersed completely.
4. Start the autoclave with lid closed till temperature reaches 80° C.
5. Drain the complete water from the autoclave after 30 mins.
6. Repeat the cycle steps from 2 to 5, if required.
7. Clean the inside chamber with clean water and wipe it.
8. Add fresh water and now autoclave is ready to use.

For best results, clean equipments every 20 cycles or once per week as needed.

It is essential to follow the equipment manufacturer's directions for cleaning and maintenance.

STERIMAXX INC

"A reliable partner in Sterilization !!!

A/55 Shankesh Bldg, Sanghvi Nagar,
Mira Bhayander Road, Mira Road (East).
Thane 401 107 Maharashtra, INDIA.
Cell: +91 9987 51 4946.

sterimaxx@gmail.com. www.sterimax.com